


Skills Set (4 Tasks)

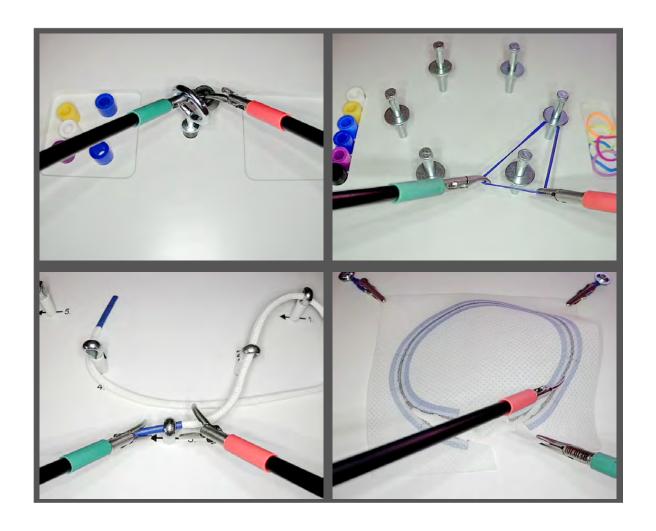

Manual

1. Pok-A-Tok

Task

Move the beads from the container on the left-hand side to the container on the right-hand side. Each bead must pass through the ring in the middle of the task board.

Preparation

Put all the beads (6 pcs) in the container on the left-hand side. Prepare two dissectors.

Procedure

Grasp the first bead with the instrument in your left hand and push it through the ring. Take the bead with the instrument in your right hand and put it in the container on the right-hand side. The task is completed as soon as all the beads are in the container on the right-hand side.

Time limit









2. Hexagon

Task

In the first step, stretch the rubber bands between neighbouring columns making a hexagon shape. In the second step, put the beads on the top of the columns.


Preparation

Fill the container on the left-hand side with beads (6 pcs). Put six rubber bands in the container on the right-hand side. Prepare two dissectors.

Procedure

Take a rubber band with the instrument in your right hand. Stretch the rubber band between two neighbouring columns using both instruments. The rubber band must be placed beneath the disk above the middle of the column. Repeat the procedure with the remaining rubber bands. The first part of the task is completed as soon as all the rubber bands are stretched between the columns forming a hexagon shape. Take a bead from the container with the instrument in your left hand. Then pass the bead to the instrument in your right hand above the middle of the task board. Put the bead on the top of a column so that it lands on the disk. The second part of the task is completed as soon as all the beads are placed on the columns.

Time limit

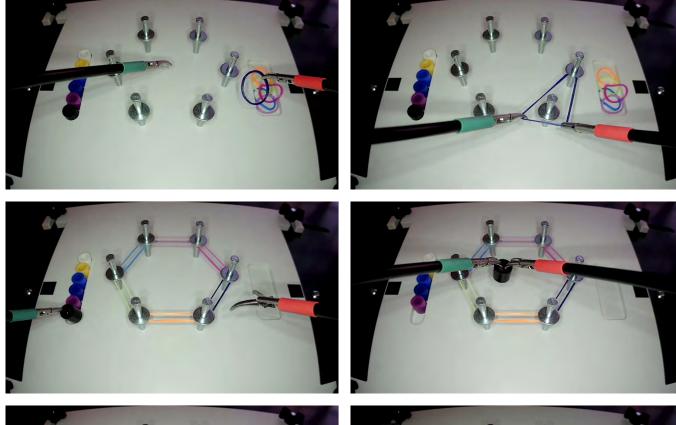



HELAGO-CZ, s.r.o., Kladská 1082/67, 500 03 Hradec Králové 3, Czech Republic, info@helago-cz.cz, www-laparoscopic-trainer.com

3. Shoelace

Task

Pull the shoelace through the eyes of all the pins according to the order numbers and the direction of the arrows.


Preparation

Cover the perforated task board with the pattern. Insert the pins into the marked holes so that the axes of their eyes correspond with the arrows. Prepare two dissectors.

Procedure

Pull the shoelace through the eye of each pin. Respect the order numbers and the direction of the arrows on the pattern. The two instruments must work together. The task is completed as soon as the shoelace has passed through the eye of the pin marked with number 5. Repeat the task with all the patterns in the order of the letters A, B, C, D.

Time limit

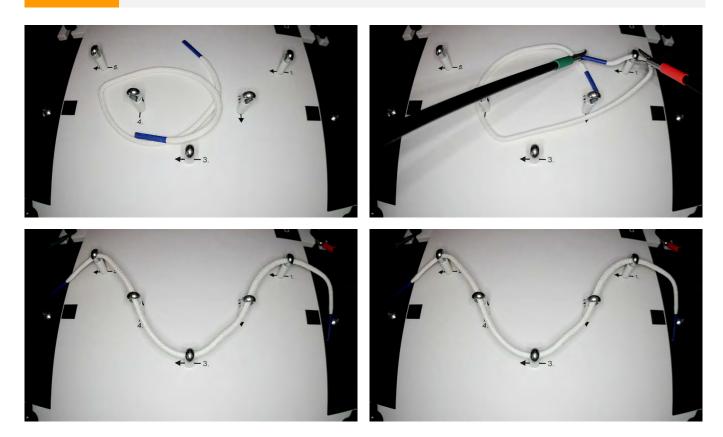

4. Precise cutting

Task

Cut out the round middle part of the cloth according to the printed pattern.


Preparation

Attach the square cloth to the board with the clamps. The cloth must be positioned right in the middle of the board. Prepare a dissector and a pair of curved scissors.

Procedure

Lift the edge of the cloth with the dissector and cut the cloth where the colour belt is interrupted. Cut out the round middle part of the cloth step by step. Cut with the scissors precisely along the black line. Hold the cloth with the dissector so that the direction of the scissor blades always corresponds to the shape of the black line. The circle is divided into eleven sections. You may enter the light blue zone once within each section. Entering the dark blue zone is forbidden. The task is completed as soon as the round middle part of the cloth is completely cut out. You may swap the instruments between your right and left hands during the procedure.

Time limit

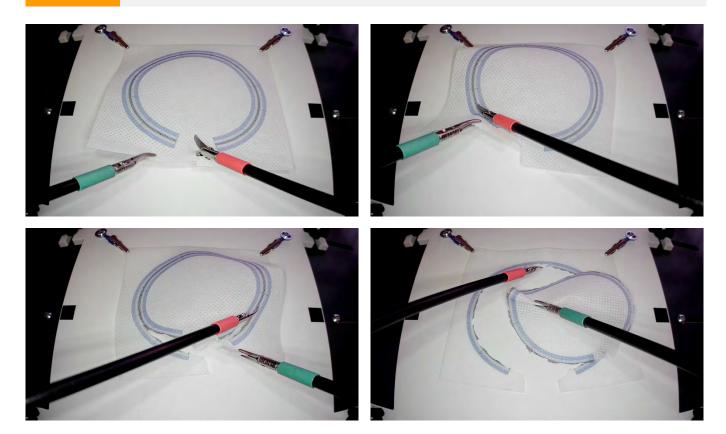

General instructions

Hang the instruments in the hooks on both sides of the task board when they are not being used.



Time measuring starts/stops at the moment when both instruments have entered/left the operating field.

FOR MORE INFORMATION PLEASE CONTACT:

HELAGO-CZ, s.r.o. Tel: +420 l JÍ Á33€Á3GJ E-mail: info@@/|æ*[Ë&: .&: